

PureComfort

Leg Compression Massager



Important safety instructions:

All the safety and operating instructions should be read, adhered to and followed before the unit is operated.



Distributed By:
Home Aide Diagnostics, Inc.
1072 S. Powerline Rd.
Deerfield Beach, FL 33442
TEL. 1-800915-0116 | www.homeaide.us


Cautions:

1. Seniors and/or new users should limit the usage time to 5 min/section. You may increase the time usage by 5 minutes per section when you feel your body is well adapted.
2. Always unplug the unit from the electrical outlet when not in use.
3. Never insert finger or object inside the cushion.
4. Never use the product in the bathroom or in wet places.
5. Store in a cool dry place.
6. Do not hold the product by the wire as this may result in malfunctioning of the product.
7. Do not sit on this cushion to prevent any break down.
8. It is advisable to use the device at least one hour after meals.
9. An adult should supervise use of the device by a child or the physically handicapped.
10. Do not try to repair the device yourself or through any unauthorized persons or agent, or it will void the warranty.
11. Should the device malfunction, disconnect it from the power source and contact the authorized agent immediately.

Description of each part



Controller

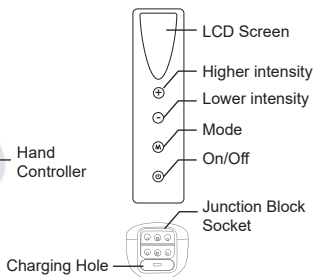

Product Specifications

Model No: SM-512DB
Product Type: Electrical Massager
for use on Calfs, Feet and Arms
Lithium Battery: 900mah
Voltage: DC5V 1A
USB line: 3ft 3in (1meter)


Charging time: 1H
Using time: 1H
Timer: 15 mins
Gross Weight: 1.1KG

Usage



1. Fully charge the remote control for 1hr prior to using the product.
2. Unplug USB cord and store for next use.
3. Connect the rubber tube from the remote to the correct side massager cuffs (i.e LEFT or RIGHT).
4. Smugly (not tight) wrap around the chosen extremety with the velcro.
5. Choose desired level or mode on remote

Controller Operation

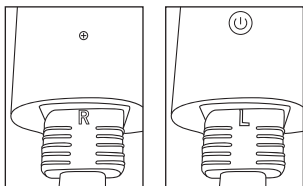

1. The silicone gas pipe plug on the cloth cover should b inserted in the corresponding hole position of the controller.

2.E



Cleaning and maintenance

Cleaning



WARNING

- Switch off the device each time before cleaning.
- **Clean the device with a slightly damp cloth only.**
- Under no circumstances may liquid enter the device or accessories.
- Do not use the device again until it is completely dry.
- Do not clean the device in a dishwasher.
- Do not use any abrasive cleaning products or hard brushes.

CARE

- Whenever the product is not in use, please store it in a cool, dry place in the original packaging. Do not place other items on top as the pressure could damage the product.

Disposal



IMPORTANT

- Batteries must not be disposed of in the household waste. They may contain poisonous heavy metals and are subject to special refuse treatment.